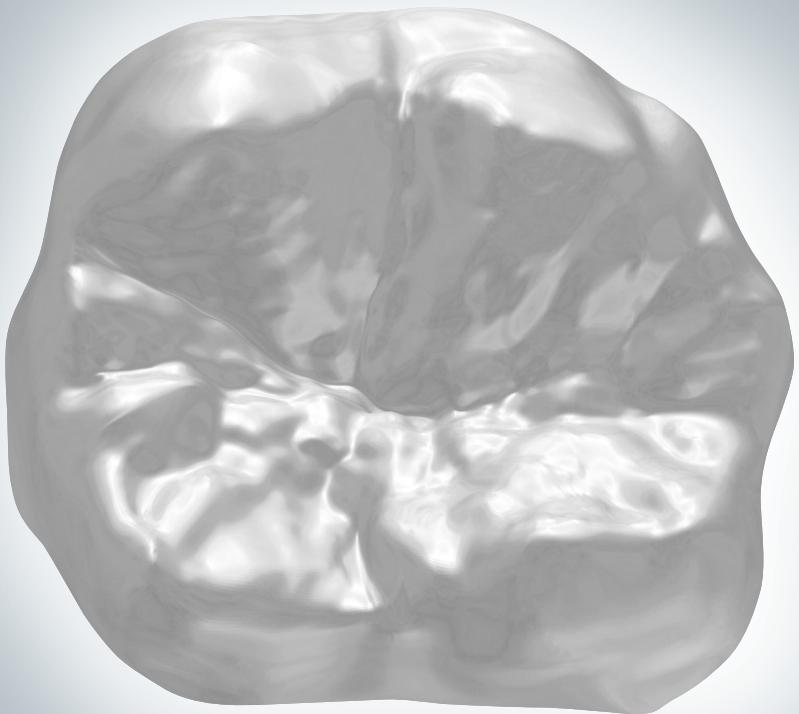
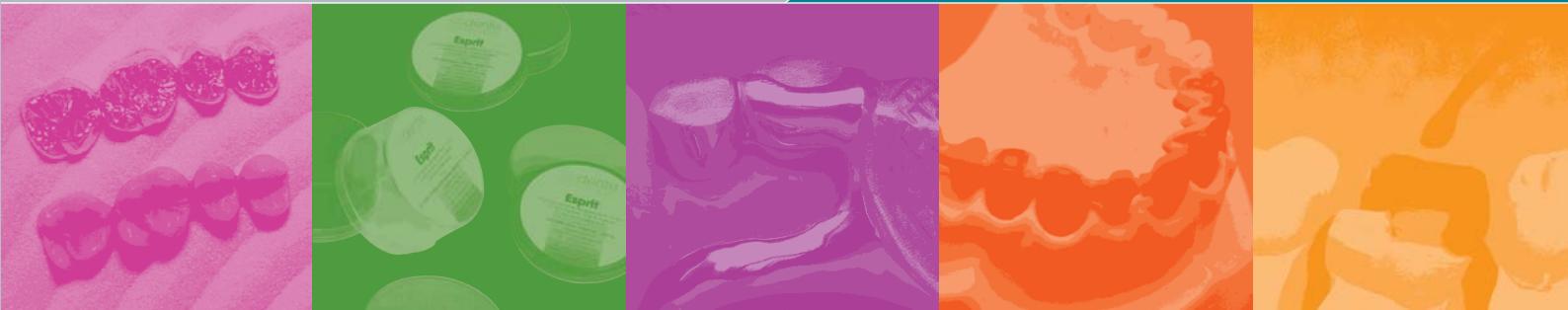
2009/10

al dente
dentalprodukte

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■ Many thanks for your interest in our products.

Our new catalogue has become once again a comprehensive list of products, which will provide you, as technicians in your daily work, with benefits in the form of reliability and time-saving.

Our products result from daily laboratory practice in close co-operation with those in the field. You therefore have the certainty of using in your lab high quality products which have been tested in practice.

Naturally, and without exception, we make our products according to DIN EN ISO 9001:2000 and EN ISO 13485. Furthermore there has been no change in our objectives:

By constant high quality, fair pricing and efficient service to be your partner in laboratory routine.

We wish you much pleasure in the perusal of our catalogue and in the use of our products.

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■ Orders should be directed to our Partner Depots in your own country. You will find the address on the reverse of the catalogue.

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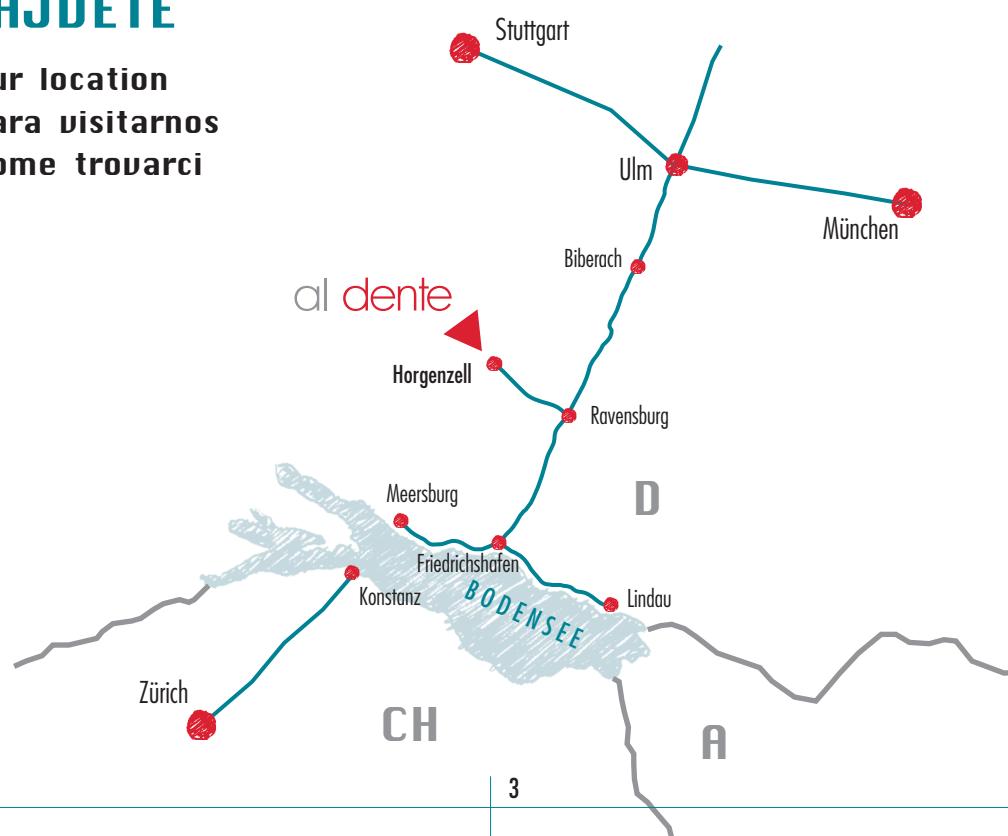
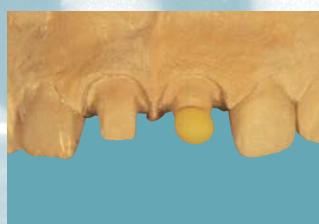
■ Rational time-saving diagnostic waxing up with **al dente** wax ups. Available in 5 natural upper and 1 lower plate form. Pure organic wax, thus also suitable for pressable ceramic systems.

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■ Rational time saving diagnostic waxing-up with **al dente** wax ups.

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■ No collar = optimum aesthetic. Anatomic form. Occlusal depth for natural fissures. Deeper cut in of labial, due to lingual/palatinal transferred connecting points.

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ADVANTAGES

► Time-saving = increase of productivity
► Wax pontics in different sizes. The individual adaption of the patterns is therefore reduced to a minimum. ► Optimum wax quality with perfect scraping and modelling qualities. ► Due to the hard wax a bending of the modellation while removing is impossible.

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
■ The ceramic layers are limited by the lingual/palatinal collar, thus enabling a timesaving and well-aimed application of the ceramic.

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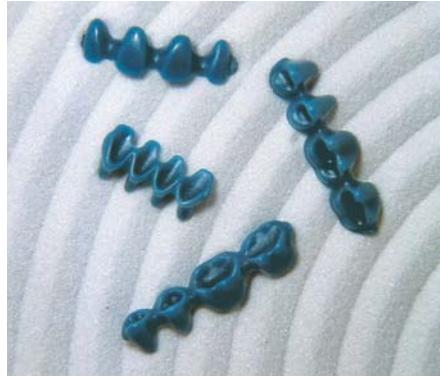■ Anatomic form. ► Wide, approximate connecting points for tightless linking. ► Reduced form variety (18 forms) = small storage = low costs. ► 2 different sizes (A+B). ► Also available as hollow pontics in the posterior area.

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■ Weight reduction of up to appr. 45%.
► Patient friendly, as low metall costs.
► Tightless construction through almost the same volume of the cast between crowns and wax pontics. ► Easiest processing. Hollow pontics are filled with ceramic and baked as usually.

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■ Formed according to bio-dynamic knowledge. ► Formed exactly according to the natural model. ► Universal size for quick adaption to the given situation. ► Special wax, with excellent modelling quality.

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Packing units

of 25 wax occlusal patterns
or range of 4 x 15 wax occlusal patterns

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

■ Perfectly matching wax quality in consistent colour. The residue-free burning-off waxes can be shaped to the model without tension. Marks can be easily seen through the transparent wax.

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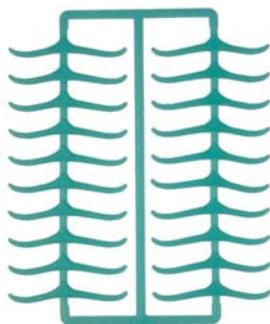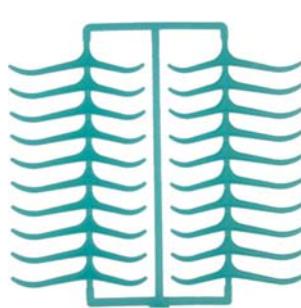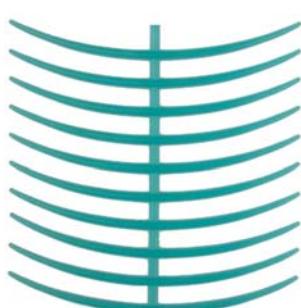
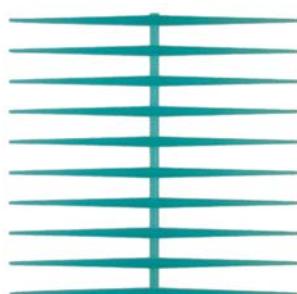
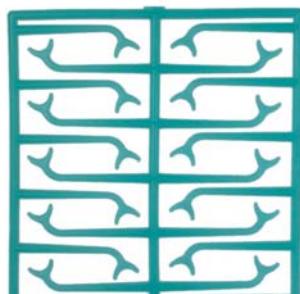
■ Anatomically shaped wax profiles
No reverse forces, free of tension
Content of pack 10 bars
Colour turquoise

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■ Self-adhesive casting wax guarantees that the underside of the acrylic is absolutely precise.

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■ The sculpturing wax has the same consistency as the wax patterns, which enables a harmonious modellation and it is not necessary to carve from hard into soft wax.

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■ For blocking out undercut ponts in frame work technique. **al dente** Blocking Out Wax adheres firmly to plaster, and is pared easily. Contains non-combustible ingredients, therefore unsuitable as casting wax.

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AVANTAGES

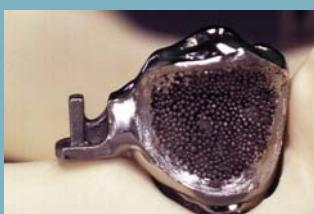
- No soldering ► No impure solder in patient's mouth ► Completely tension free. ► Ca. 30 min. time saving ► Reduced cost ► Improved quality

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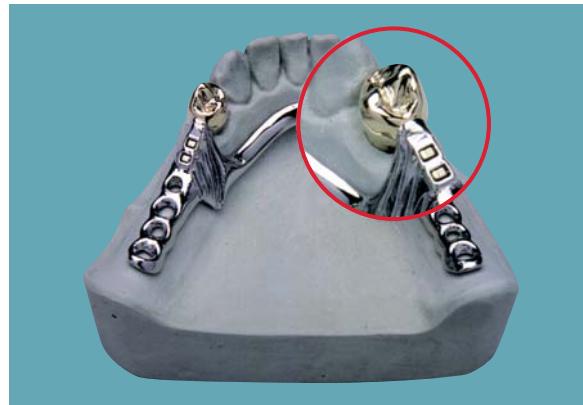
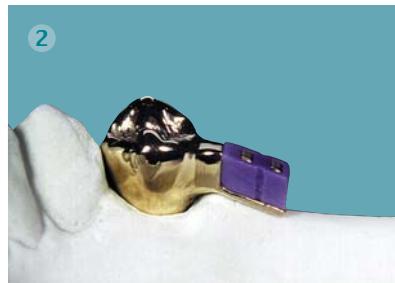
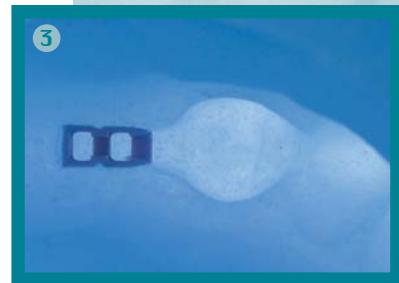
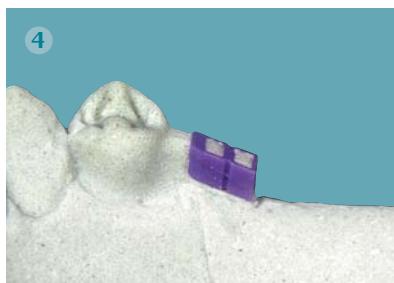
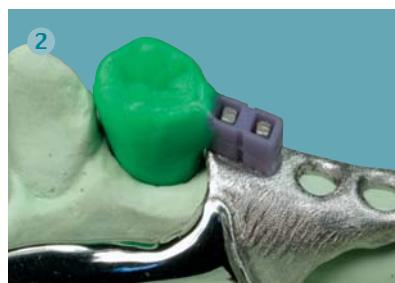
■ Pro-Bond is a 2-component adhesive, specifically for connecting retainers units, or for the connections between anchors, attachments to the partial denture. Pro-Bond is usable with hot polymerization. Melting Point 250°C – Flash Point 150°C.

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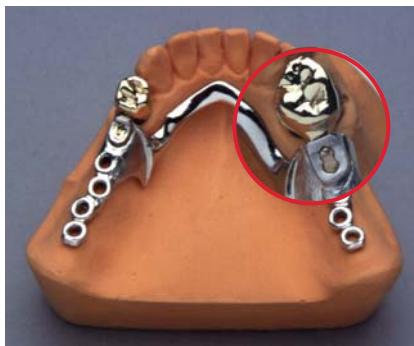
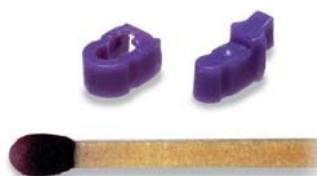
■ al dente laser weld fixings for tension- and distortion-free laser weld fixings. Through the ready made adhesive elements you obtain in shortest time an easily reproduced secure joint with consistent quality.

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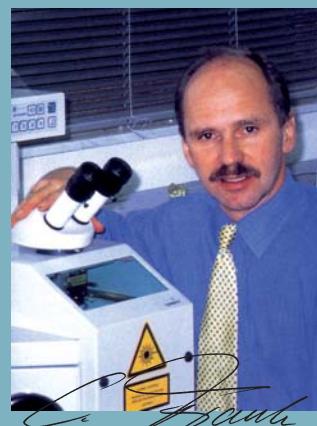
ADVANTAGES

- Ready-made adhesive element ↗ Reproducible quality ↗ Proof for quality assurance. ► No distortion or tensions through locating welding ► Time-saving ↗ Reduced cost ► No duplication part necessary

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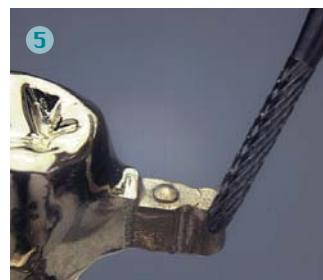
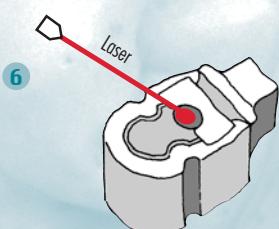
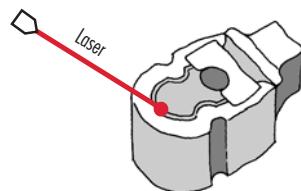
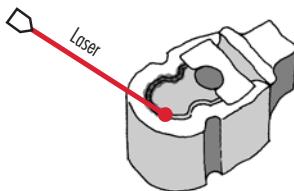
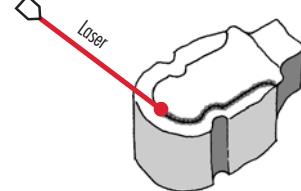
■ Sculpturing waxes are an important, often under-estimated element in the absolute success of dental technology. In the manufacture of our sculpturing waxes we lay greatest emphasis on the use of high quality raw materials. **al dente** sculpturing waxes are developed in co-operation with notable international dental technicians, tested, and are most compatible with each other in their characteristics. Convince yourself!

The following table gives an over-view of the specific application areas of the individual waxes.

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■ Perfection in wax! Would you have thought that these are wax modellings? In co-operation with Wolfram Schultis we have succeeded in developing a set of 12 differing waxes which are perfectly matched in colour to the natural tooth. The Real Art modelling wax set is thus ideally suited to the production of demonstration items, true-to-nature Wax ups or to the practising of layer and inset techniques.

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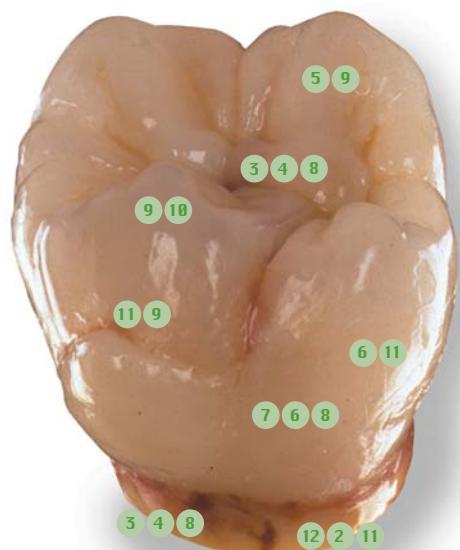
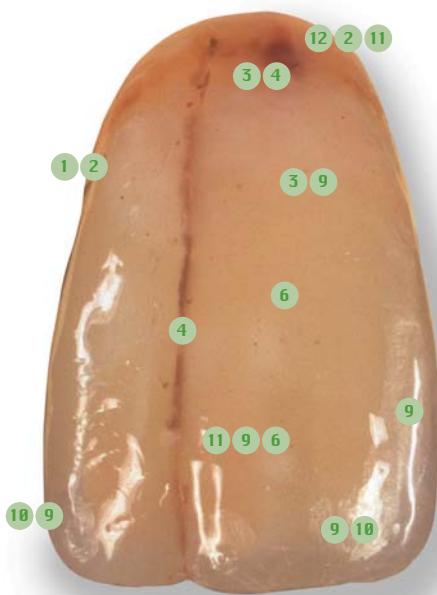
■ Pro-Mod sculpturing wax, the ideal allround sculpturing wax for the whole crowns and bridges technique with excellent modelling qualities and smallest contraction. Pro-Mod always leaves a smooth surface. The high steadfastness and the low point of becoming solid make a precise pulling of the wax possible, e.g. for any waxing-up technique. Pro-Mod burns without leaving traces.

Pro-Mod is available in 2 different hardnesses (regular/hard (H)).

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Art. Nr. 02-1000H
Art. Nr. 02-1010H
Art. Nr. 02-1020H
Art. Nr. 02-1030H

PRO-MOD C KRČKOVÝ VOSK

Pro-Mod C cervical wax | Pro-Mod C cera cervical
Pro-Mod C cera cervicale

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■ A cervical wax, such as every technician desires: lead-inert, tension-free, easy flowing, non-smear, non-friable and completely exact end result. **New:** Cervical yellow. Why always fish about in the dark? Because of the light transparent colour the mark of the cervical edge shows through the wax. Thus precise optical control of the edge is assured.

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■ Esprit – Sculpturing wax for universal applications

Esprit establishes new standards in wax technology. Esprit brings together all the qualities now required of modern sculpturing waxes by technicians. Esprit can be drawn out excellently in order to build up a mass or to form edge mouldings. Esprit is extremely shape-retentive and contraction-resistant.

Because of its internal stability Esprit can be cleanly pared, leaving a smooth surface hardly requiring any further work. Occlusal stops and contact points are reproduced in the cast true to the original because of the reduced inner elasticity of Esprit. New opaque pastel shades assure a soft harmonious contrast to the plaster model and are very pleasant to the eye.

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■ The new organic "ash-free" diagnostic wax is adapted to the Vita shades A-B-C-D. Ideally suited for wax-ups and castable ceramic modellations.

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■ Colors by **al dente** is the new modern modelling wax in current neon colours. Colors distinguishes by an excellent rigidity and it is quick to cool. It's the ideal wax for all modelling applications. It also suits for castable ceramic wax-ups.

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Sculpturing wax for castable ceramic systems

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■ The castable ceramic technique poses special requirements to sculpturing waxes. **al dente** VKS waxes are free of opaque constituents, so that undesired reactions with casting investments are avoided. Through high stability and low melting range **al dente** waxes are the ideal waxes for gnathological waxing-up techniques. Tips and edges can be drawn and built up exactly, **al dente** waxes are suitable for all current castable ceramic systems such as Cergogold, IPS Empress, Optec OPC, Dicor or others.

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■ Combination technique poses differing requirements to the waxes used. For paring the wax should not be too hard and brittle. Then for milling, a hard wax is wanted. Uniting these basic different requirements in one wax is almost impossible. Therefore we have adapted our milling wax to the various areas of requirement.

F - 0 = extra hard (milling)
F - 1 = hard (milling and paring)

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■ These waxes were specifically developed in their properties and aimed at the requirements of inlay technique. The colours brown and beige provide an excellent contrast and make possible modelling which is pleasant to the eye. Use **al dente** Aesthetic-Inlay waxes, slightly opaque, for gold casting techniques and without opaque (transparent) ingredients for full ceramic techniques. **al dente** Aesthetic-Inlay waxes are extremely form-stable and contraction resistant. Feather edges remain precisely in the cast.

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■ Shape-stable, distortion-free first layer with inlays or part crowns. Best paring and modelling properties. Transparent – suitable for all current castable Ceramic systems.

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■ Tropical is a special wax used in very high ambient temperatures. The special features of Tropical are a higher melting point with a short cooling-off period at the same time. Tropical is also an ideal wax for all castable ceramic systems. It burns without residue and has excellent modelling and paring qualities.

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■ For polishing wax models and removal of excess wax.

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■ After dipping the wax coping there are 2 possible ways of obtaining an exact marginal edge:

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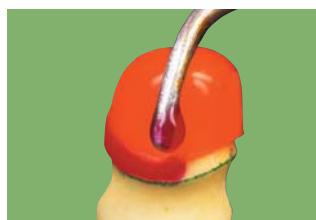
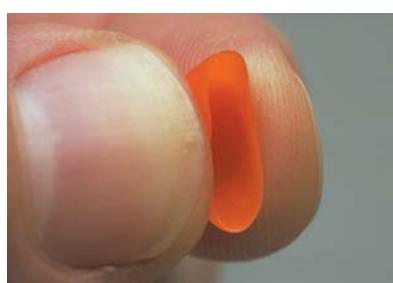
■ Flexi-Dip, the elastic dipping wax for achieving precise fitting wax copings with reproducible wall thickness and excellent scraping and cutting qualities. The high elasticity of the wax makes deforming of the coping impossible, giving best fitting and easy adaptation of the casted crowns. The wax is free of opaquing elements and burns without leaving traces. Ideal dipping wax temperature ca. 87° Celsius gives ca 0,35 – 0,4 mm cap wall strength.

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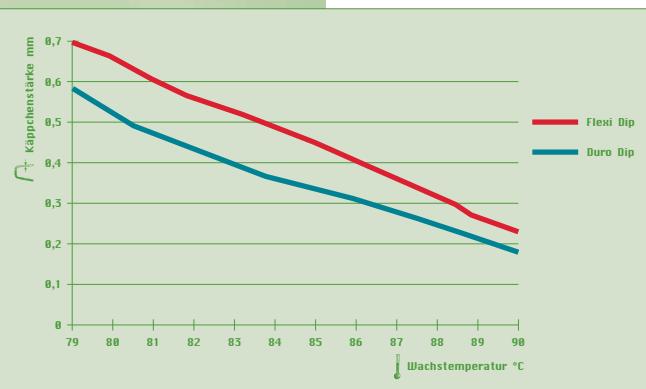
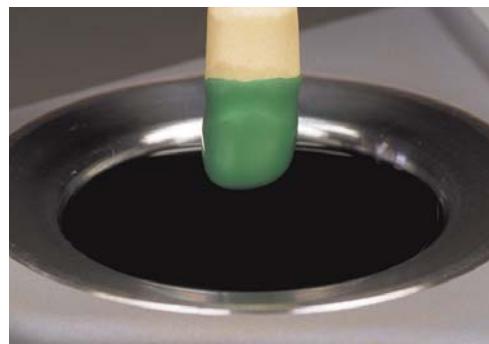
■ Duro Dip, the new hard elastic dipping wax is characterised by its high precision and facility of modelling. Ideal dipping wax temperature ca. 84° Celsius ca. 0,35 – 0,4 mm. cap wall strength.

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Art. Nr. 02-3100
Art. Nr. 02-3110

QUICK-DIP KAPNIČKOVAČ S REGULACÍ TEPLITY

Temperature controlled dipping pots

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■ Precise temperature control prevents over heating of the dipping wax. Available in standard model or digital model for precise setting and monitoring of wax temperature.

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■ Which technician working with ceramics hasn't encountered this problem? Parts of the ceramic shoulder break off when lifting the crown from the die. Some systems try to solve this problem by a mixture of ceramic powder and wax. Using this method the ceramic is not sufficiently compressed resulting in porosities and poor fitting of the ceramic shoulders.

Our solution:

The shoulder mass is applied and compressed in a proven way. Only after this process you apply the ceramic shoulder wax by **al dente** onto the ceramic shoulder. The ceramic shoulder is stabilized by the wax layer applied and can be taken off the die without any risk. Thereby an optimal fit is reached and the ceramic shoulder does not contain any porosities. The purely organic ceramic shoulder wax by **al dente** presents a low point of building drops, diffuses easily into the ceramic mass and burns without leaving residues (minimum firing temperature 780°C).

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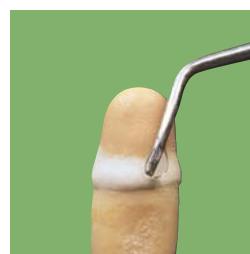
■ Sticky permanently soft wax for positioning of wax pontics.

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■ Special Sticky wax with very high adhesive power. Ideal for the fixing of break repairs, teeth, front walls and in solder models. Because of its high hardness any distortion of the fixed work is impossible. **al dente** Sticky wax can be simply removed by use of a steam jet.

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■ For blocking out defects and cavities in plaster stumps. Pro-Mod B adheres to all plaster surfaces, is easily modelled and scraped and because of the high melting point does not unite with other waxes such as, for example, dipping wax.

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■ Sculpturing wax in a big 110g block for universal applications. Excellent scraping and modelling qualities. Available in 2 transparent colours, for castable ceramic systems, and 2 opak colours for metal systems. Very low contraction for a tension free modelling. Burns out without leaving traces.

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■ al dente wax wire is free from resting forces and can be shaped without pinches or cracks. Available in 2 different hardnesses.

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ADVANTAGES

- Reshaping of the ball to its technically most ideal shape of flow: the nozzle. This shape has the advantage that the alloy being injected swirls less.
- A more homogenous texture without cavities is the result.
- Easiest waxing of the sticks by soaking of the waxing stick. Aperture diameter of 3mm and the concave shape of the reservoir on the waxing side assure good accessibility and thus enable metal economy and easy removal after casting.

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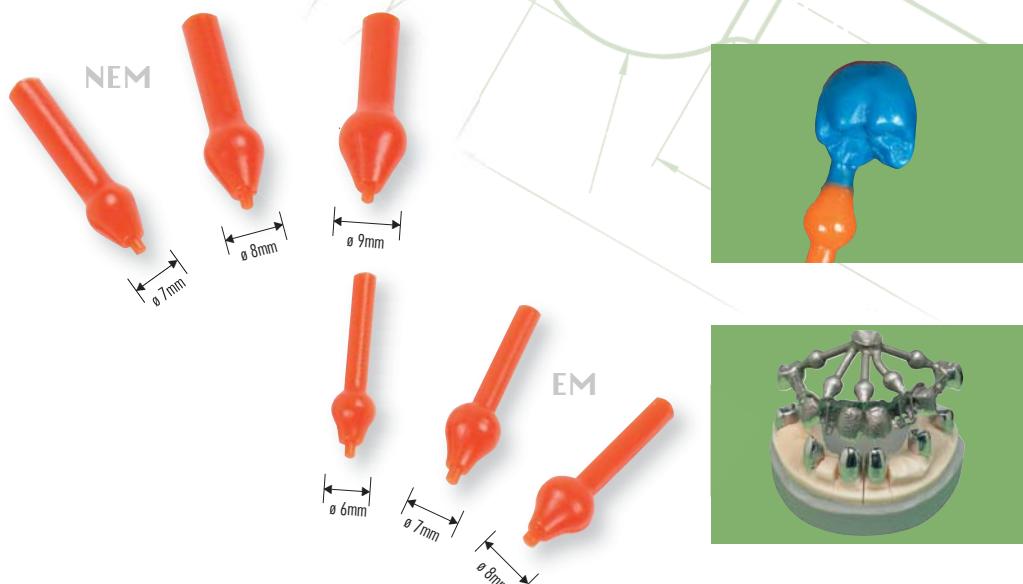
■ Most ideal shape of flow resulting in a more homogeneous casting structure without cavities or shrinkholes. The **al dente** casting sprues “EM” with a sprue’s head diameter of 6–7–8 mm are used for precious metal alloys.

”EM“ licí kanálky pro ušlechtilé kovy
Casting Sprues ”EM“
Bebederos de colado ”EM“
Canali di colata ”EM“

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New: al dente casting sprues “NEM” with a sprue’s head diameter of 7–8–9 mm and an increased cross-section of the sprue are especially used for vacuum pressure castings, non-precious alloys and precious-metal reduced alloys.

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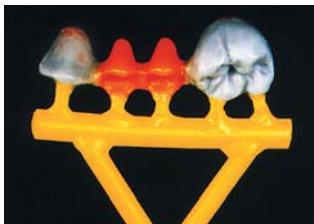
■ Round or square? You must decide that. Either way, the specially hard adhesive wax of the Sticky sticks facilitates application and prevents distortion of the cast when removing the modelling from the mould.

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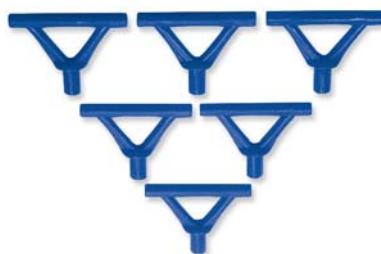
■ Set of triangular casting sprues. Burns without leaving residue, produced without any inorganic components.

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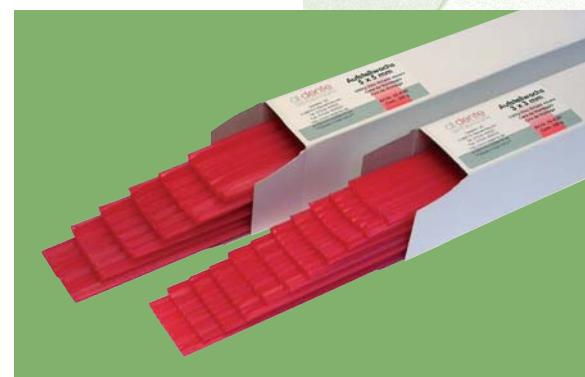
■ For a quick and secure positioning of the teeth. Colour red. Removes easily with warm water.

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Set-up plate waxes | Set-up planchas de cera
Set-up cere a disco

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■ By the harmonious combination of bees waxes, micro waxes and paraffins **al dente** Set-up satisfies highest requirements. The high plasticity prevents tearing when shaping. **al dente** Set-up can be modelled and cut extremely well. When shaping **al dente** Set-up does not bend back and has high adhesive strength without sticking. For differing ambient temperatures **al dente** Set-up is available in 2 qualities (regular/hard).

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■ Pre-prepared bite surfaces for the time-saving manufacture of precision bite moulds. The anatomic shape with basal ridge makes possible a slight waxing of the bite surface on the mould. We supply three different hardnesses of bite surfaces.

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DeguDent
A Dentsply International Company

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■ In co-operation with the DeguDent GmbH we have developed special laser-optimised modelling waxes for CAD/CAM systems. With **al dente** CAD/CAM modelling waxes the matching accuracy of machine produced workpieces is significantly increased.

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Art. Nr. 02-5010

■ The wax for the **al dente** CAD/CAM Wax Disc is especially adjusted to the dental CAD/CAM technique. Thereby crowns and bridges can be virtually formed, milled and finally casted the conventional way. The wax is excellently machinable and burns without residue. This results in smooth casted superficies. These optimized features of the wax give way to even very delicate forms excluding shrinkage or distortion of the milled object. The wax – stable in volume – permits absolutely exact margin finishings and fits. Up to 30 units can be milled out of one disc.

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■ Water-removable die spacer suitable for scan techniques. Recommended for all other current CAD/CAM systems. After use the Aqua Spacer is easily removable under running water (see page no. 44).

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Art. Nr. 03-4170

ARCTIC SERIES ŠTĚTEC NA KERAMIKU

Brush Arctic Series | Pincel Arctic Series | Penelli Arctic Series

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■ Kolinsky brush for ceramists of extraordinary quality, ergonomics and design. Uniquely designed ceramic brush houses metal ball bearings dynamically form a more defined bristle point after shaking or snapping the brush. The tip of the brush itself is hand made from exquisite high quality Kolinsky bristles. The included slipcover avoids a deformation of the bristles when the brush is in a glass of water.

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ZIM Adam Schäuf, Head of Art Department

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■ „Ice tool I“ – (size 8) is ideal for applying important amounts of ceramic portions as well as for incisal or occlusal layers.

„Ice tool II+III“ – (size 6+4) suits especially for finest layers or for the use of stains.

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■ The triangular ceramic brush is handmade using the finest quality red sable Kolinsky. The tight and exquisite red sable bristles offer a sufficient liquid reservoir. By using the fine and elastic brush tips the ceramic mass can be more precisely and evenly applied. The handle is ergonomically designed and the triangular shape adapts well in your hand and it allows you to work without symptoms of fatigue. The triangular shape of the handle also prevents the brush from rolling off the table.

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■ Triangular ceramic brush made of synthetic bristles of high quality. The tip of the brush is very tight and pointed. The approved ergonomic triangular shape of the handle prevents the brush from rolling off the table and allows you to work without symptoms of fatigue.

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■ The new ceramic mixing palette Blues continues the successful story of the Ceram-X mixing palette. The features and characteristics of blues were increased. Due to the membrane's technique the moisture content of the ceramic mass is always maintained on exactly the same level. The results: best properties for the model work and a brilliant outcome after the firing process.

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■ Due to the excellent strength of the ceramic mixing liquid compounds, the ceramic mass can be applied in increasingly moist layers. This leads to an even setting after the modelling, whereby the ceramic mass sinters consistently. More brilliant colours are the result. The drying time is approx. 4 minutes.
► Excellent strength of the ceramic mass ► Less shrinkage of the ceramic
► More brilliant colours ► More constant drying of the mass ► Suitable for opaque powders and shoulder masses

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■ The **al dente** ceramic texture marker can be used to accurately reproduce the shine and texture of the surface between ceramic surfaces and the natural neighbouring teeth. It is easily noticed if the ceramic in layers blends harmoniously in the general view. The ceramic texture marker is available in gold and silver colours. The marker comes premixed and ready for immediate use. No primer or similar products are necessary. Flask with brush – no soiling of your place of work. The ceramic texture marker can be easily removed with water or with a steam jet.

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■ Extremely high effect of isolating plaster against the ceramic. Very thin coating of isolation. The isolating effect stays for a long period of time. No discolouration of the ceramic. Leaves no residues.

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■ **al dente** Aqua Spacer can be easily removed from the stump under running water. Thus solid ceramic crowns can be applied in the shortest time. A further advantage is that the self-colour of the ceramic is not affected by intensively coloured Die Spacers. Complete assortment comprises of Aqua Spacer, thinner, special isolating material, hardener, dipping wax, cervical wax and modelling wax for the solid ceramic technique, all in a practical plastic container.

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STEPS IN COATING THE STUMP

► Expose limit of area for preparation and mark ► Only apply hardener (O μ) to the area limit for fixing prep. limit. ► Apply Pro-Die or Pro-Lux Spacer approx. 1mm above the preparation limit. ► Allow stump to dry for a few minutes
Attention: Stump must be dust free and dry ► Stump must not have a base of cyano-acrylates ► Apply light curing spacers as thinly as possible

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► Pro-Die Hardener (O μ) for hardening, sealing and protecting of dies and margins. Does not build up. Is completely absorbed into the plaster.

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► Pro-Die Spacer, the Die Spacers, which were developed specifically for the requirements of dental technology. Pro-Die Spacers are distinguished by:

- Fast drying times
- High final hardness
- Resistance to evaporation
- High coverage ability

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■ Have you ever seen anything like it? A Die Spacer which shows you when work can be resumed on dies. Chameo, the new Die Spacer from **al dente**, changes colour from red to violet during drying.

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■ Pro-Lux light curing Die Spacer, the new light curing Die Spacer by **al dente** with qualities which convince:
► Pro-Lux hardens within about 90 seconds under all current light hardening apparatus (Light colour 78). ► Pro-Lux hardens glass-hard but not brittle. ► Pro-Lux is resistant to all steam jets. ► Pro-Lux is also eminently suitable for sealing the model base under a bridge part. The layered ceramic no longer breaks when lifted but can be easily taken from the model.

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■ Pro Seal wax sealing is applied to the modelling before embedding (drying time about 5 minutes). By the application (thickness approx. 5 μ) of Pro-Seal, small scratches and unevennesses are levelled out. Contact points can be targeted in applying. The modelling is polished. The upper surface is homogeneous after casting, thus reducing work time.

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■ A thin layer on the mould or matrix of an attachment part allows one to identify the position of irregularities, thus simplifying adjustments.

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■ Fluid, thinly flowing pigment (3μ) for marked identification of flaws in crowns, inlays and rubbing points. Contact Marker marks all materials, such as metal, plaster and plastics. Contact Marker covers dependably with only one application and can easily be washed off again with water. Contact Marker is available in 3 colours.

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■ By using the occlusal spray by **al dente** trouble spots can be quickly found.
► Economical use ► Second nozzle for an extra fine spray ► easily removable with water ► exact distinction of trouble spots ► The gas propellant is environmentally safe and contains no fluorocarbons ► nonflammable

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■ Iso-Pro soaks completely into the plaster upper surface and thus does not deposit any interfering film. Iso-Pro is characterised by its extremely good isolation effect with only one application. Iso-Pro has no alcohol content and is water soluble, so is neutral in relation to sculpturing wax and investment material. Iso-Pro is also obtainable in the convenient 250ml refill bottle.

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■ Upper surface release of wax objects and silicon impressions. Good moistening between wax and investments or between impression material and plaster model is assured. Castings and plaster models become noticeably smoother and more bubble-free.

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Instant adhesive with patented cap

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■ Who is not familiar with this? Nozzle blocked! With **al dente** instant adhesive this belongs in the past. The patented turnlock cap prevents conglutination of the nozzle reliably. **al dente** instant adhesive can always be re-opened and closed securely. The adhesive can thus also be used to the last drop. **al dente** instant adhesive is available in two consistencies.

Universal – Application: gluing of pins, plaster, plastics

Capillary thin-flowing – Application: sealing, hardening, gluing

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Art. Nr. 03-1000
Art. Nr. 03-1020

UTEŘINOVÉ LEPIDLO - RATIONAL

Instant adhesive - rational | Pegamento - racional
Cianoacrilato - rational

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■ Economical instant adhesive with excellent adhesive power for all materials used in the lab.

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■ Light-curing tray material with excellent physical properties. Easy to adapt on the model through lead-inert behaviour of the composite material. Highly shape retentive and break-resistant. Pre-prepared upper jaw plates in the colours pink or transparent.

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■ Light-curing material with very good modelling and adhering properties. **al dente** Blockout can be polymerised with any UV-Curing Light in common usage. After hardening **al dente** Blockout is edge-stable and can be subsequently worked on.

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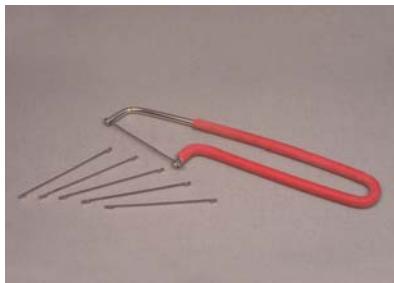
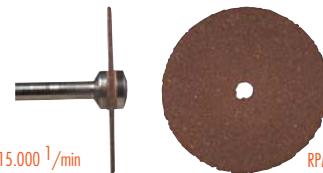
100 pcs

Art. Nr. 05-1005

- Area of application ceramic, quality metal and model casting.

- Disk pro chróm-kobaltové slitiny
- Separation disc for chrome cobalt
- Discos separadores para cromo cobalto
- Disco separatore per cromo-cobalto

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- Separation disc for chrome cobalt and NEM alloys with glass fibre reinforcement. Very long life and very abrasive.

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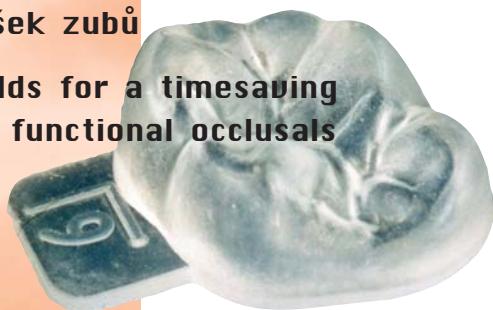
Silicon moulds for a timesaving creation of functional occlusals

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
■ Flexiform is filled with modelling wax. As soon as it begins to gel place the Flexiform mould on the stump. Close the articulator and let the wax cool down. Remove the Flexiform, the occlusal is ready. Time saving minimum 5 minutes.
With Flexiform it is also possible to create occlusals in ceramic and in acrylic.

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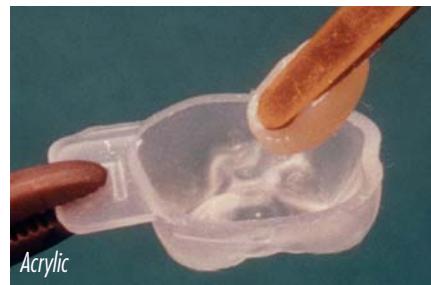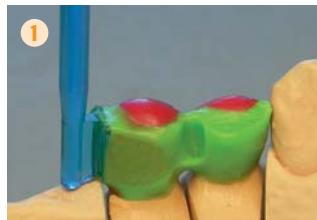
■ The vertical Rod-Attachment Kit consists of plastic burn out male patterns with an extension used as a paralleling mandrel, plastic riders and seating tool. The plastic rider helps absorb some of the occlusal force, offers better frictional retention than metal-to-metal systems, and is replacing with a quick and simple procedure requiring no soldering.

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Rod Attachment Sortiment | Assortment | Surtido | Assortimento

7 patrici + 7 matrici + 1 vtlačovací tyčinka
7 patriss + 7 matrix + 1 insert tool
7 patrices + 7 matrices + 1 lápiz de introducción
7 patrici + 7 matrice + 1 perno d'applicazione

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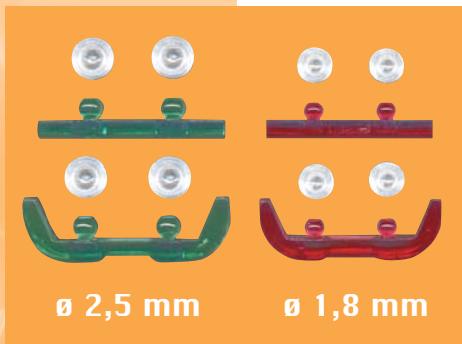
■ **al dente** ball anchor systems are inexpensive and easily to process attachments. The male is made of fully combustible plastic and can be used with all dental alloys. The female is made of permanent elastic plastic and reversible. The females are available in 4 different degrees of friction.

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■ The **al dente** ball anchor vertical has a ball diameter of 1,8 mm. Additionally it is possible to place optional a horizontal extension with integrated milling arm.

Application:

- Extracoronal attachment
- Bar retaining

**Ball anchor vertical | Anclaje esférico vertical micro
L'attacco a sfera verticale**

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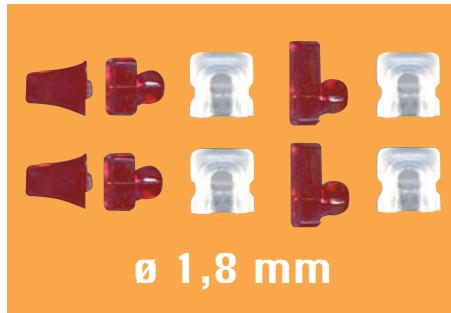
■ al dente ball anchors horizontal for overdentures or for bar-connector systems. With the help of an aluminium disk (Art. Nr. 08-0038) the attachment can be processed also resiliently.

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Art. Nr. 08-0016

www.al-dente.de

The screenshot shows the homepage of al-dente.de. At the top, there's a navigation bar with links for Impressum, Kontakt, Handel, and Download. Below the navigation is a large image of a smiling person's teeth. A sidebar on the left contains links for News, In der Presse, Termine, and Newsletter Abo. The main content area has a heading 'Willkommen' and a paragraph of text. At the bottom, there's footer information about programming, design, and login.

This screenshot shows a sub-page on the al-dente website. It features a large image of a smile, a navigation bar with links for Impressum, Kontakt, Handel, and Download, and a sidebar with links for News, In der Presse, Termine, and Newsletter Abo. The main content area contains text about the company's collaboration with dental technicians and its commitment to quality, mentioning ISO 9001 certification and its global reach.

This screenshot shows a products page for 'Modellierwachse'. It includes a navigation bar with links for Impressum, Kontakt, Handel, and Download. The main content area features a large image of a smile, a sidebar with links for various modeling wax types like Tiefzieh-, Polymerisations-, and Frästechnik, and a product image of a dental model.

www.rsb-duna-dental.de

This screenshot shows the website for rsb-DUNADENTAL. It features a blue header with the company logo and navigation links for Home, Firma, Produkte, Tiefziehtechnik, Polymerisation, News, Kontakt, and Impressum. The main content area displays images of dental equipment and a message welcoming visitors to their partner for clinics and laboratories.

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Ahesive bondings	13
Aesthetic-inlay wax	27
Aqua-Spacer	44
Aqua-Spacer for scan techniques	39
Attachments	54
Ball anchor	55
Binding joints	13
Binding joints for laserwelding technic	16
Bite surfaces	37
Blocking out material	51
Blocking out wax	12
Blockout wax C+B	33
Brush Arctic Series – Ice tool	40
CAD/CAM waxes	38
Casting wax	12
Ceramic isolation	43
Ceramic mixing liquid	42
Ceramic mixing palette Blues	42
Ceramic shoulder wax	32
Ceramic texture marker	43
Cervical wax	22
Chameo Die Spacer	46
Clasps profiles	10
Colloidal graphite	47
Colors – sculpturing waxes	24
Contact Marker	48
Diagnosticwax	24
Die Lubricant	45
Dipping wax	29
Dipping pots Quick-Dip	31
Debubbler	49
Duro-Dip dipping wax	31
Esprit sculpturing wax	23
Flexi-Dip dipping wax	30
Flexiform	53
Fixing wax	33
Hollow pontics	9
Inlay wax	27
Instant adhesive	50
Iso-Pro Die Lubricant	49
Kolinsky ceramic brush	41
Light curing tray material	51
Milling wax	26
Occlusal spray	48
Paralleling tool	15
Partial technique	10
Partial wax	12
Prefabricated wax pontics	6
Pro-Bond 2-component adhesive	15
Pro-Die Hardener	45
Pro-Die Spacer	45
Pro-Lux light curing Die Spacer	46
Pro-Mod sculpturing wax	22
Pro Seal wax sealing	47
Real Art modelling wax	20
Rod-Attachment	54
Saw blades/Saw bow	52
Sculpturing waxes	18
Sculpturing waxes for CAD/CAM Systems	38
Sculpturing waxes for Cercor-System	38
Separation discs	52
Set-up plate waxes	37
Set-up wax	36
Sprues with reservoir	35
Sticky sticks	36
Sticky wax	33
Synthetic ceramic brush	41
Triangular casting sprues	36
Tropical wax	28
Undercut wax	27
Universal sculpturing wax	34
Universal-wax adhesive	10
VKS wax	25
Water removable Die Spacer	44
Wax brush	28
Wax occlusal patterns	9
Wax Retentions	11
Wax ups waxveneers	5
Wax wire on spools	34
Wax sealer	44

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al **dente** dentalprodukte GmbH